


MASK FILTER INSERTS


WARNING: ENSURE MASKS ARE PREWASHED BEFORE USE
WASH OR SANITISE HANDS PRIOR TO INSERTING MASK FILTER.

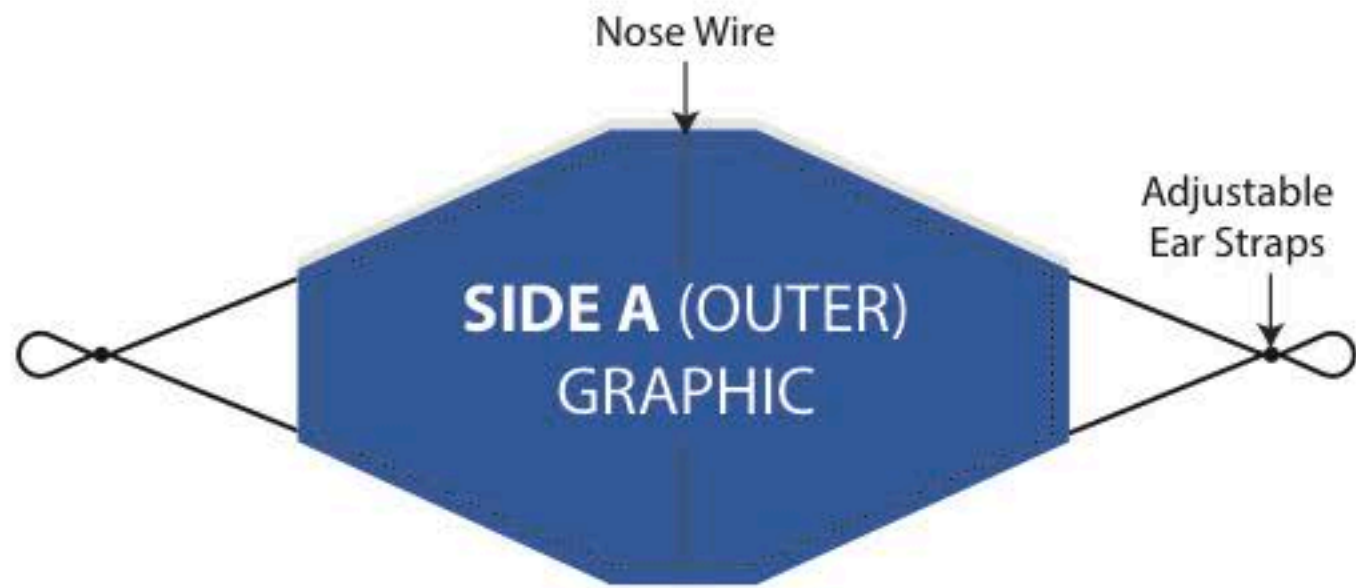

↑ ↑ ↑

**FILTER INSERT
(OPENING)**


STEP 1:
Invert mask

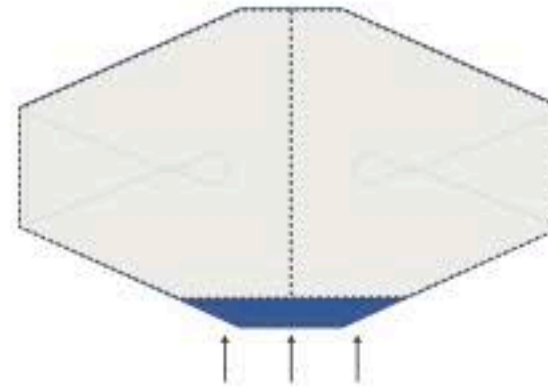


STEP 2:
Position fingers
within mask




STEP 3:
From inside inverted mask centre
fingers on seam and nose wire

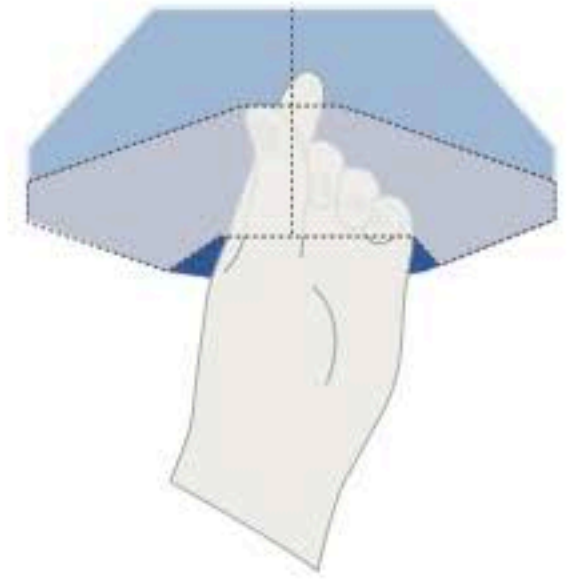


STEP 4:
Place and pinch filter
in between forefinger & thumb

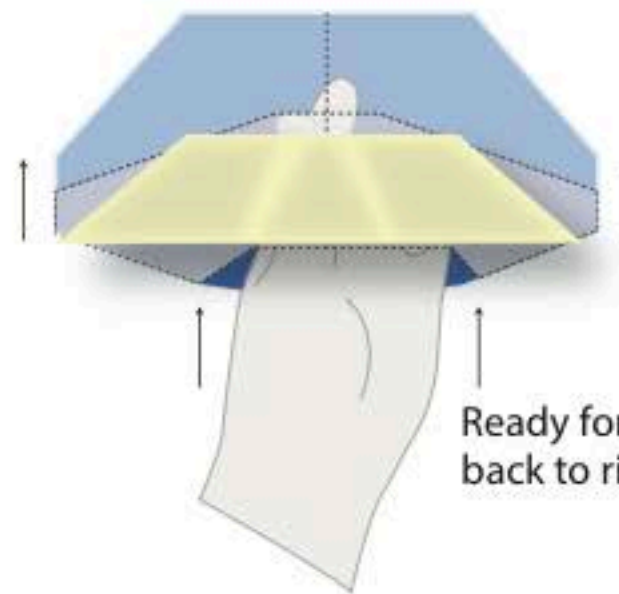

STEP 5:
Firmly hold onto filter
pullover from opening

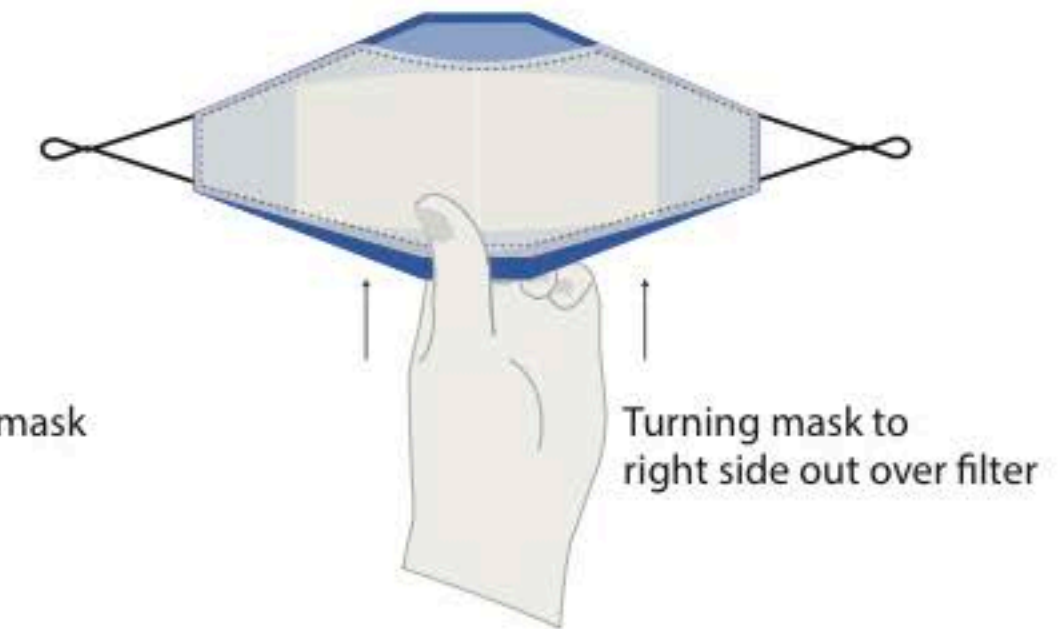

STEP 6:
Still pinching
filter down

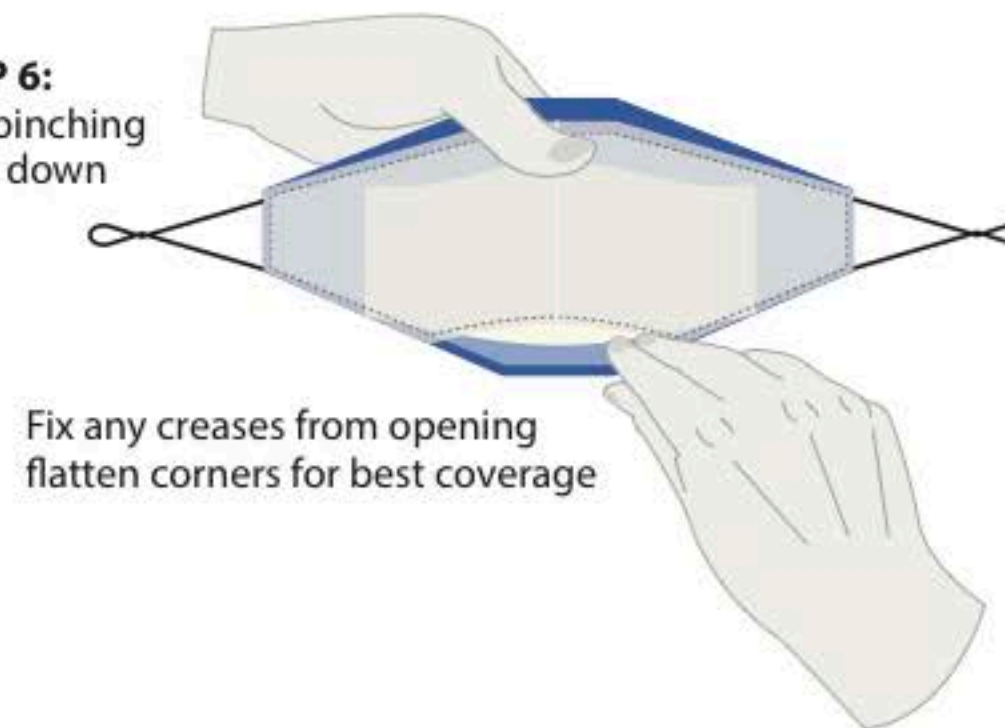


STEP 7:
Ready to wear mask



FACE MASK - USAGE GUIDE


WASH BEFORE USE

Wash before first use.
Launder masks daily.
Always remove filter before washing.

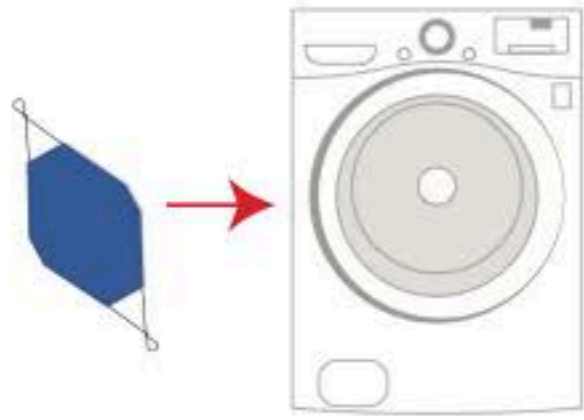


WASH YOUR HANDS

Always wash your hands with soap and warm water for at least 20-seconds before and after handling the mask.

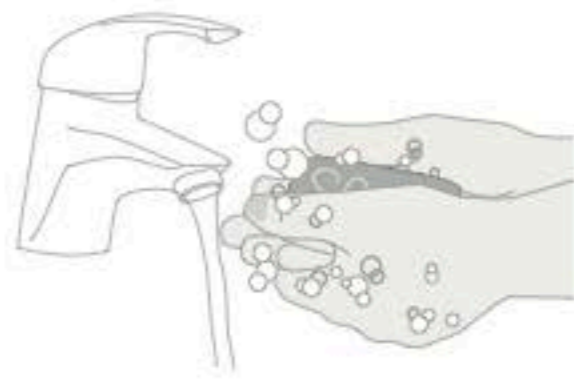

REMOVE FILTER

STEP 1
Change the filter after one day use.

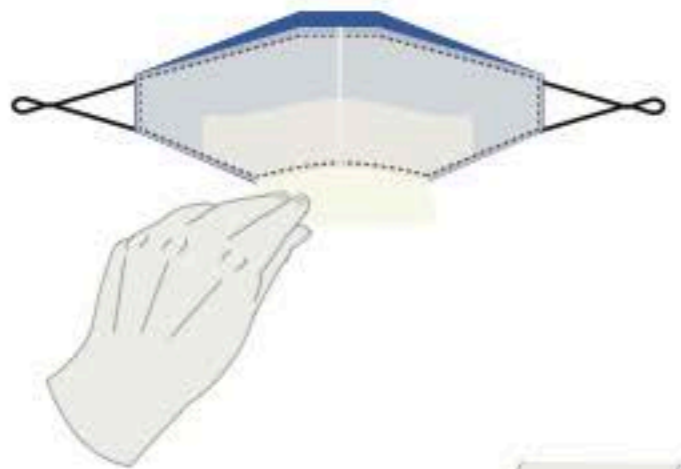


STEP 2
Discard filter immediately after removing it from mask. Do not leave filter lying around.




FITTING MASK

STEP 1
Cotton lining side facing you.

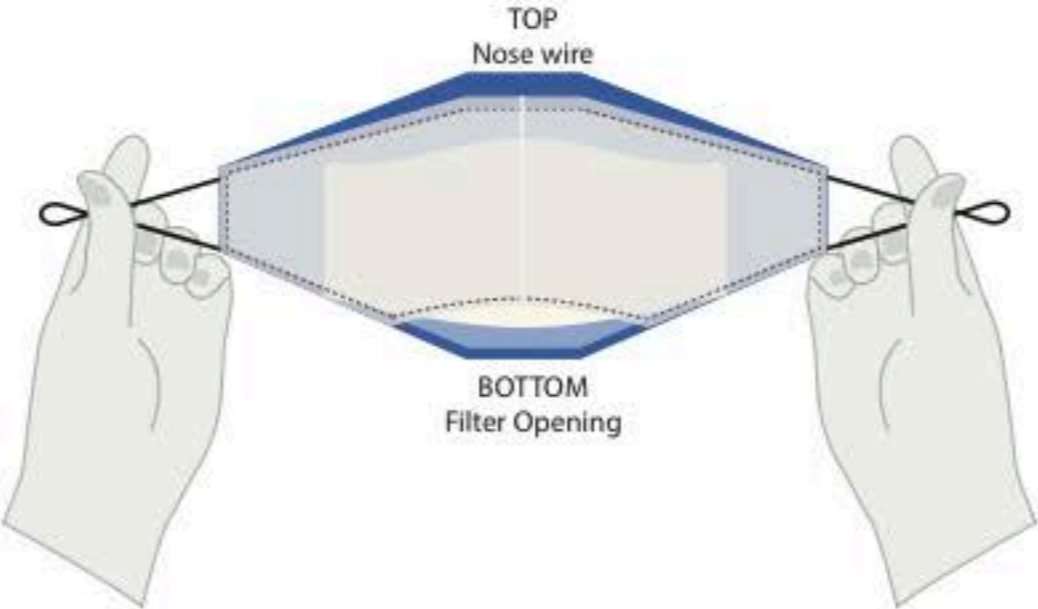

STEP 2
Starting from left or right ear. Place ear strap over ear, one at a time. Position mask to cover nose and mouth fittingly.

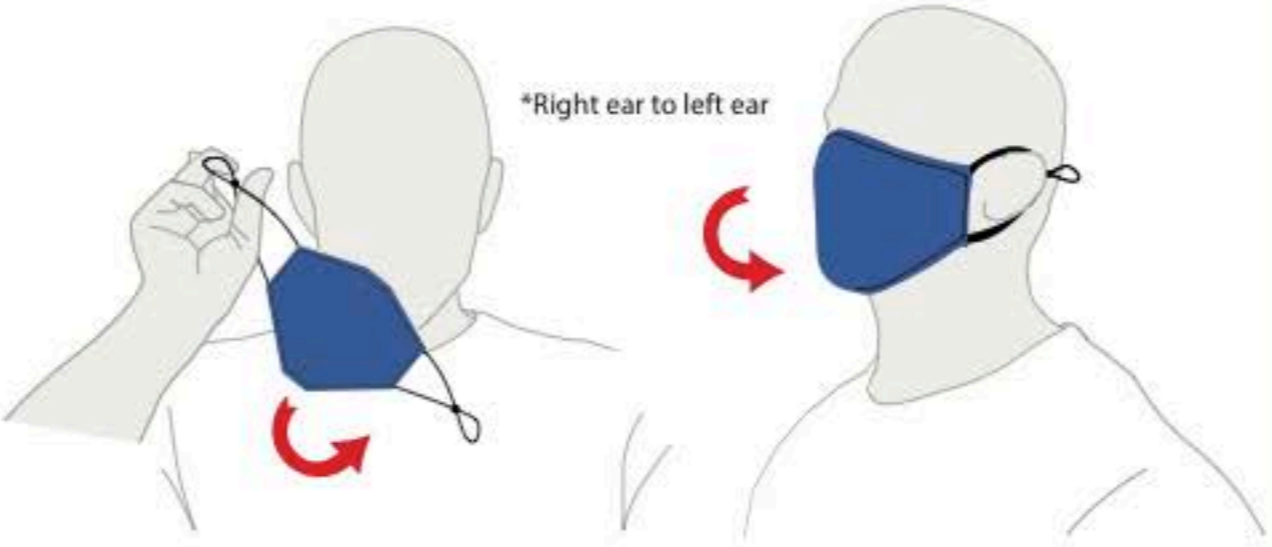


STEP 3
Pinch the shapeable aluminium nose wire to help form a better seal under the eyes and around nose bridge.



REMOVING MASK

STEP 1
Only touching the ear straps, unhook first ear then next.

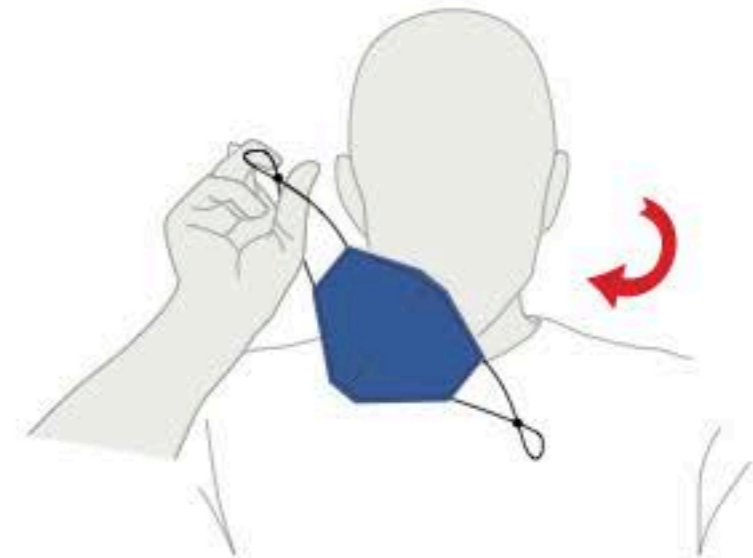

Never touch the mask as you risk contaminating yourself. Keep hands off face, wash hands after every removal.



At intervals when no need for mask, with sanitised hands fold mask for later use protecting inner lining.

